



Quality,
Innovation
and You.




SAFETiHEEL

Contact-Activated Safety Heel Stick


SAFETiHEEL, an innovative quality product brought to you by NSP Medicals. The team at NSP has developed a single-use contact-activated safety heel stick designed specifically with Infants in mind. The device incorporates a world's first "usage indicator" and it offers a gentle and safe blood sampling device for newborns, infants and babies.

Whether you are a trained medical professional or a concerned mother, SAFETiHEEL's unique and patented activation mechanism, ensures a consistent and controlled incision to deliver a pain and trauma free experience. Afterall, the team at NSP are parents too.

SAFETiHEEL's blades are always encased. Each device is deactivated after use and cannot be reused thereby eliminating risks of cross contamination.

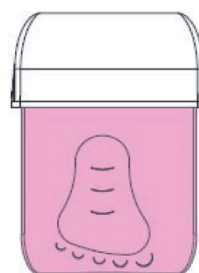

SAFETiHEEL

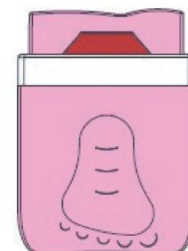

Red indicator,
before
activation



After
activation

Directions for use:




Once the site on the heel has been selected, clean the area with an antiseptic swab and allow to air dry. Remove the protective cap.



Press the SAFETiHEEL firmly against the baby's heel in one continuous motion to activate it.



Use a gauze to wipe away the first drop of blood. Use the desired capillary tube or collection vessel to obtain the needed specimen.

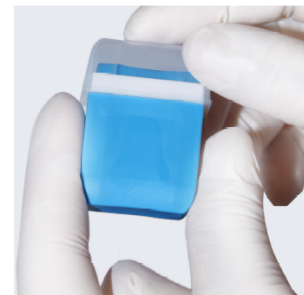

Put the cap back on to the SAFETiHEEL. Discard it into a biohazard sharps container.

Key Features & Benefits

Contact-Activation

Ensures consistent penetration depth to enhance user experience by eliminating the likelihood of having to repeat the procedure.

Innovative

One step activation, does not require pre-loading. All models comes with a unique usage indicator which tells instantly if product has been activated. Truly simple to use and reduces training time.

Gentle

All blades are ultra thin, providing the finest, cleanest surgical incision. With a guided blade movement, it ensures utmost comfort and minimal pain.

Safety

All SAFETiHEELs are Gamma Sterilized and packed in individual blister packs. The automatic blade retraction system prevents it from being reused. This eliminates the risk of needle stick injuries and prevents cross contamination.

Quality

SAFETiHEEL remains sterile until the blister pack is opened. The surgical blades, which are concealed within a safety cap prevents exposure and contamination. SAFETiHEEL is manufactured under strict quality international standards that meet regulatory requirements.

Technical Data & Ordering Information

Product Description	Use	Product Number	Colour	Depth (mm)	Width (mm)	Packaging
Micro-Premie	< 1000 grams	SH - 065	White	0.65	1.40	50/box, 500/case
Premie	Low birth weight	SH - 085	Yellow	0.85	1.75	50/box, 500/case
Newborn	Birth up to 6 months	SH - 100	Pink	1.00	2.50	50/box, 500/case
Infant	6-24 months	SH - 150	Blue	1.50	3.00	50/box, 500/case